



Lung Ultrasound

ASSESSMENT OF OBSTETRIC PATIENTS FOR COVID-19 RELATED COMPLICATIONS

Aly Youssef, MD, PhD

Department of Obstetrics and Gynecology, Sant'Orsola Malpighi University Hospital, University of Bologna, Bologna, Italy

Carla Serra, MD, PhD

Department of Organ Failure and Transplantation, Sant'Orsola Malpighi Hospital, University of Bologna, Bologna, Italy

Ultrasound has been used to aid in diagnosis of common chest pathologies due to its high sensitivity and specificity. Cost-effective, easy to use, safe and repeatable, the lungs can be rapidly assessed at the point-of-care. This makes ultrasound the ideal tool to use in obstetric patients with suspected COVID-19 infection to minimize interaction with other healthcare providers.

PERFORMING LUNG ULTRASOUND



- Convex: Start in OB or Abdomen Preset
- Linear: Start in MSK or Small Parts Preset
- Phased: Start in Abdomen Preset




- Scan patient sitting, supine or lateral position




- Start at depth of 11-13 cm (average adult)
- Position focus at level of pleural line
- Low or no persistence to visualize the pleural sliding
- CrossXBeam™ CRI imaging off to evaluate B-lines

Visit <http://www.volusonclub.net/emea/latestnews?id=2693> for more information.


TRANSDUCER POSITIONS

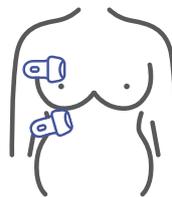

Anterior

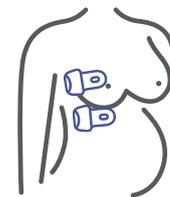

Lateral/Axillary

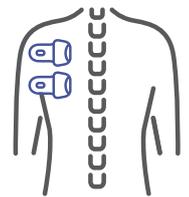


Posterior

- Lungs should be examined bilaterally
- Each lung is divided into 6 segments
- Examine lungs in both axial and sagittal planes

NORMAL FINDINGS

PLEURAL LINE

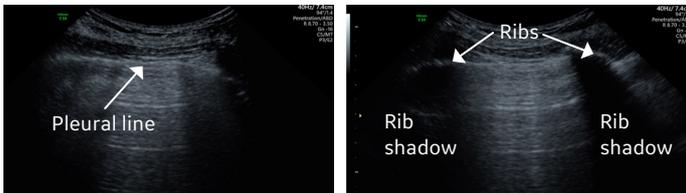


Axial

Sagittal

- Appears as a thin hyperechoic line
- Can be seen in normal lungs
- Demonstrates motion “sliding” on respiration

A-LINES

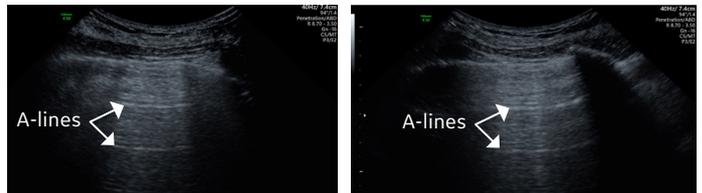


Axial

Sagittal

- Do not demonstrate motion
- Evenly spaced horizontal lines under the pleural line
- Correspond to normal reverberation artefact of the pleural line

FINDINGS IN COVID-19

B-LINES

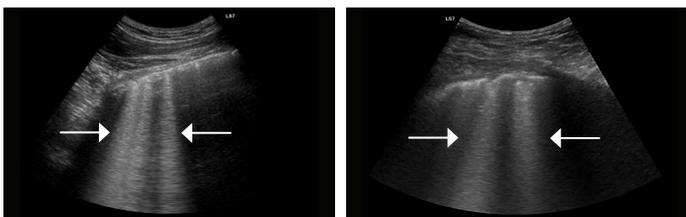

- Start at pleural line and radiate vertically through the image
- A-lines are no longer visualized
- Indicates moderate lung aeration loss resulting from interstitial syndrome

COALESCENT B-LINES



- Also called “white lung”
- B-lines that arise from pleura and radiate through image demonstrate coalescence

LUNG CONSOLIDATION



- Lungs acquire a (hypoechoic) tissue-like echotexture
- Caused by massive aeration loss