

# Application News



## Voluson E series DICOM storage on DVD/CD or USB device

On the Voluson E6, E8 and E8 Expert BT09 it is possible to store patients on a CD/DVD or USB device in DICOM format. Also there is a possibility to store DICOM SR.

**DICOM**  
Digital Imaging and Communications in Medicine

**DICOM SR**  
SR=Structured Reporting  
The role of Structured Reporting is linking data like text and measurement to the images.

### Storing Patients in DICOM format



Select review on the Keyboard


Select the patients you want to store in DICOM via the archive, it is possible to access the archive via the touch screen or with the arrow on the monitor.



By pressing CTRL or Shift on the keyboard and then set simultaneous you can select multiple Patients. (like selecting files on a PC)



After selecting the patient select **Export**. Or by selecting it on the screen or on the touch screen.

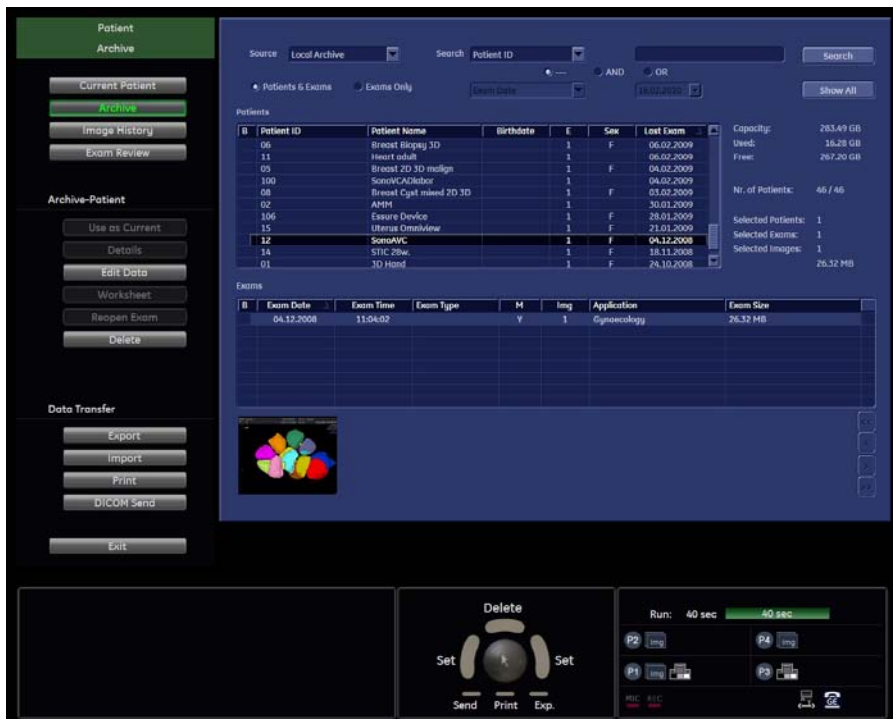

After selecting export following screen appears:



On this screen you select:

- Save in (with drop down menu): DVD/CD or USB device
- File name
- Save as type (select DICOM)
- DICOM settings you want to use. (if you are already connected to a PACS device you can select this device settings otherwise use the one called VE8 or VE6
- You can select Include SR

Select save. The patient data is saved as DICOM format.

